


Hema•Strip™ HIV-1/2

A single-use, self-contained, closed system for the collection, processing and analysis of a whole blood, serum or plasma sample for the detection of antibodies to HIV 1 and 2.

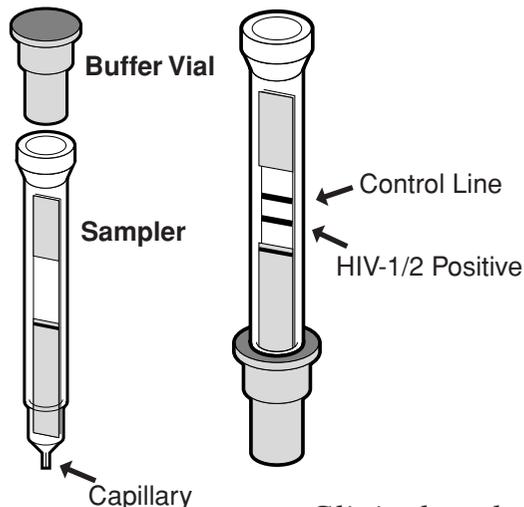

Hema•Strip™ HIV is ideally suited for field or point-of-care testing.

- No additional equipment or supplies needed.
- Valid test performance is indicated by a pink/purple control line.
- Presence of antibodies against HIV 1 or 2 is indicated by a second pink/purple line.


Components Include:

- 1 Sampler with test strip
- 1 Buffer vial
- 1 Safety lancet
- 1 Bandage
- 1 Desiccant
- 1 Disposable rack per kit


Clinical study performance data available upon request

*SIMPLE FINGERSTICK WHOLE BLOOD OPERATING PROCEDURE:


1. Prick finger using lancet.

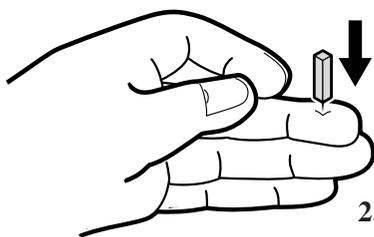


2. Touch blood drop with capillary tip of sampler.

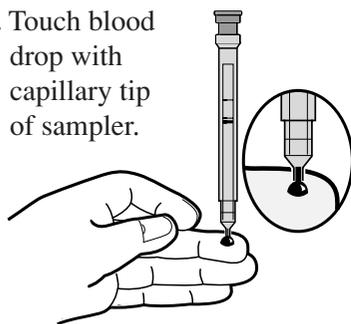


3. Remove buffer vial from top of sampler.

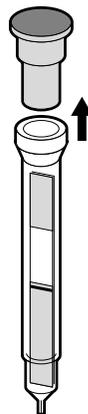

4. Push capillary tip of sampler completely into buffer vial.

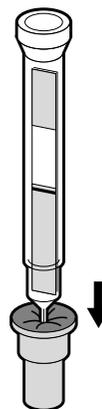

5. Read results in minutes.

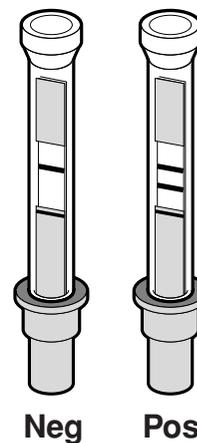

Manufactured by Chembio Diagnostic Systems, Inc. under an exclusive license from Saliva Diagnostic Systems, Inc.

For product and distributor information, contact:

**Chembio Diagnostic
Systems, Inc.**

3661 Horseblock Road, Medford, New York 11763

Tel: (631) 924-1135 Fax: (631) 924-6033

E-mail: info@chembio.com, Website: www.chembio.com

For export use only. • This product not cleared for use in the U.S.

*Please refer to the product insert for complete usage instructions

Rev 1.0, 0702