
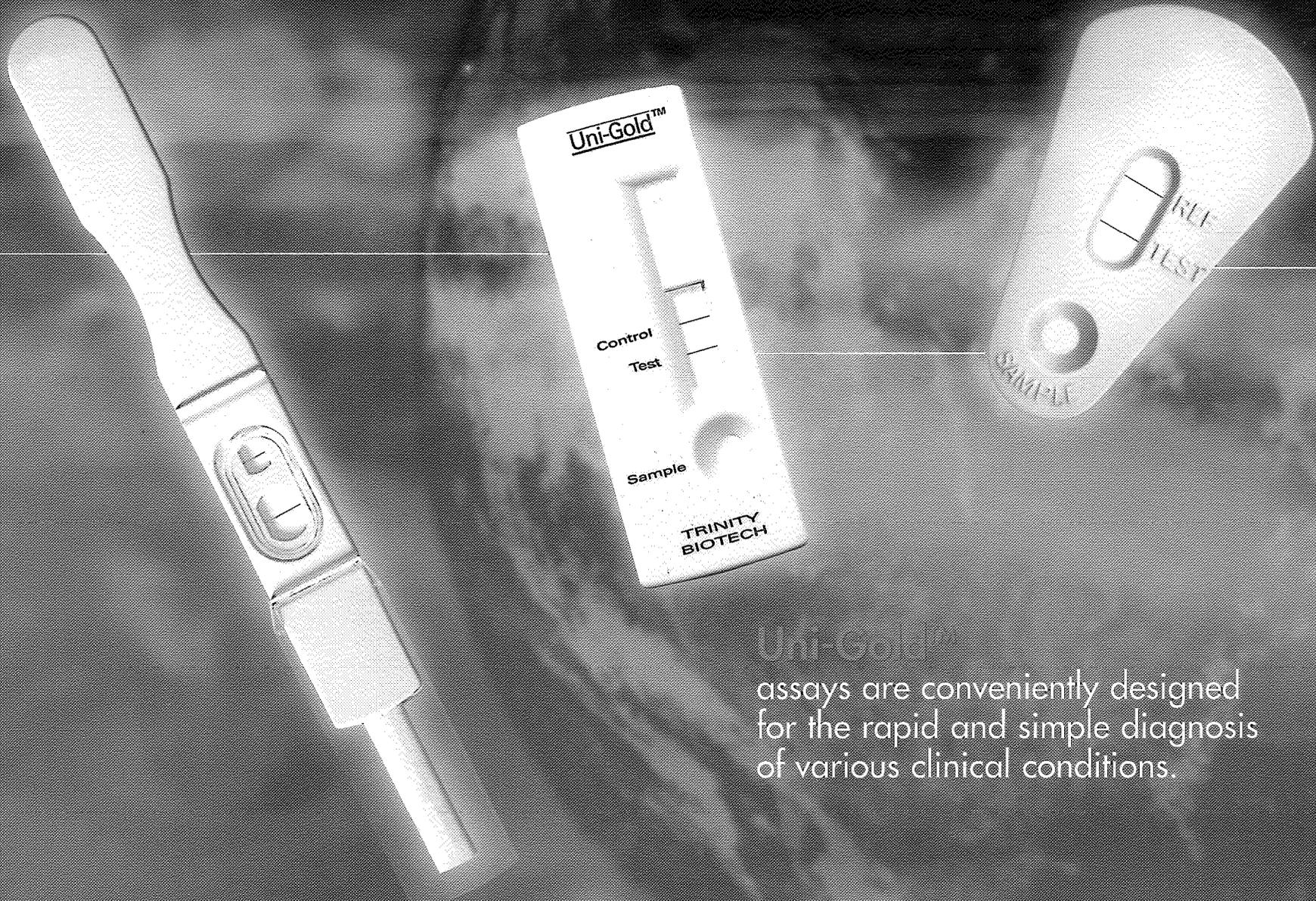
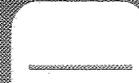


INTERPRETATIONS

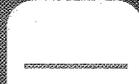

Uni-Gold™ assays are conveniently designed for the rapid and simple diagnosis of various clinical conditions.

<p>All of these results can be considered positive. The presence of a red line in the TEST area of ANY intensity illustrates a positive result.</p>	<p>Control  Test  + Positive</p>	<p>Control  Test  ++ Positive</p>	<p>Control  Test  +++ Positive</p>	<p>Control  Test  ++++ Positive</p>	<p>Control  Test  +++++ Strong Positive</p>
<p>• The presence of a red line in the CONTROL area of ANY intensity illustrates that the test has been performed correctly.</p>					
<p>Negative The absence of a red line in the TEST area indicates a negative result.</p>	<p>Control  Test  Negative</p>	<p>Control  Test  Negative</p>	<p>Inconclusive The absence of a red line in the CONTROL area should be considered inconclusive. Please refer to the individual pack insert for further instruction.</p>	<p>Control  Test  Inconclusive Test</p>	<p>Control  Test  Inconclusive</p>

Distributed by: Trinity Biotech plc, Ireland
Home page: www.trinitybiotech.com
Tel: (353) 1 295 5111
Fax: (353) 1 295 7438

• The interpretations do not apply to either the Uni-Gold™ LH or Drugs of Abuse assays.