

Virocheck HIV Rapid Test

For use with whole blood, serum or plasma
Store Kits: 15 - 30° C



- Check kit before use. Use only items that have not expired or been damaged.
- Bring kit and previously stored specimens to room temperature prior to use.
- Always use universal safety precautions when handling specimens. Keep work areas clean and organized.


This outline is not intended to replace the product insert or your standard operating procedure (SOP).



1. Collect test items and other necessary lab supplies.



2. Remove device from package and label device with the client identification number.

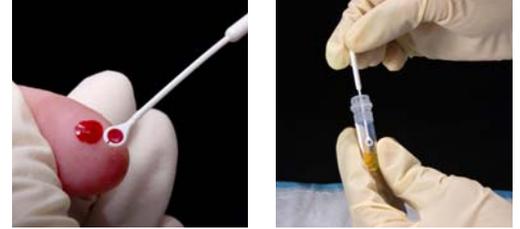

3. Collect 5 µl of specimen using new disposable loop.



4. Transfer the specimen by touching the specimen well with the loop.



5. Add 5-6 drops of the diluent buffer in a drop-wise fashion to the specimen well.



6. Wait 20 - 30 minutes before reading the results.



7. Read and record the results and other pertinent info on the worksheet.

Virocheck HIV Rapid Test Results


Positive

2 lines of any intensity appear in **both** the **control** and **test** areas.



Negative

1 line appears only in the **control area** and no line appears in the test area.

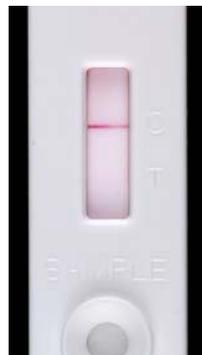

Invalid

No line appears in the **control area**. Do not report invalid results. Repeat test with a new test device even if a line appears in the test area.

