

HIV 1/2 Stat-Pak


A rapid qualitative screening test kit for the detection of antibodies to HIV 1 & 2 in human serum, plasma or whole blood

HIV 1/2 Stat Pak is ideally suited for field or point-of-care testing.


- No additional equipment or supplies needed.
- Valid test performance is indicated by a pink/purple control line.
- Presence of antibodies against HIV 1 or 2 is indicated by a second pink/purple test line.

Clinical Study Performance Data available upon request


*Step By Step Guide

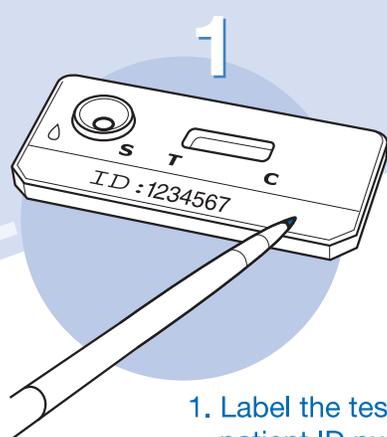

1. Label the test with patient ID number.

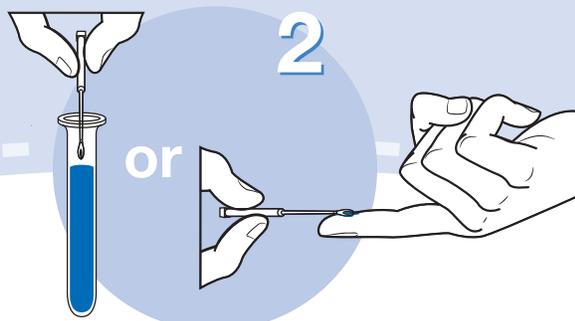


2. Collect sample using the 5µl loop provided.

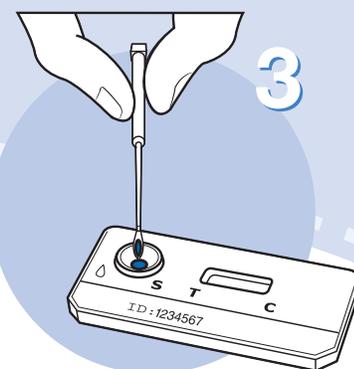


3. Touch loop to the center of sample well, holding vertically.

4. Slowly add 3 drops of buffer, holding vial vertically.

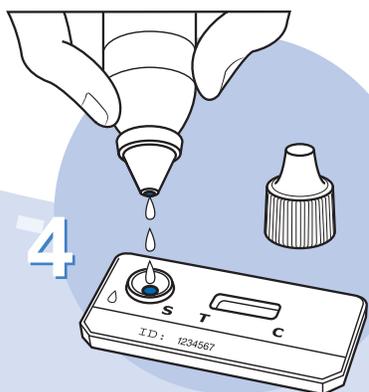

5. Read the test results at 10 minutes.

+ Positive - Negative Invalid

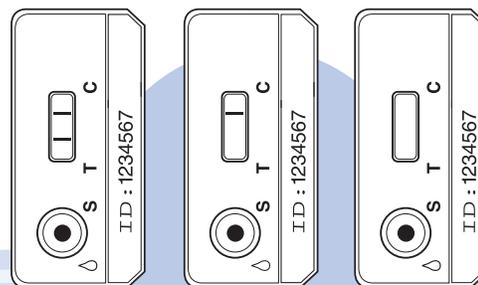

For product and distributor information, contact:

**Chembio Diagnostic
Systems, Inc.**

Chembio Diagnostic Systems, Inc.
3661 Horseblock Road, Medford, New York 11763
Tel: (631) 924-1135 Fax: (631) 924-6033
E-mail: info@chembio.com, Website: www.chembio.com

*Please refer to the product insert for complete usage instructions
Catalog # HIV101: 20-test kit; HIV133: bulk tests.

For export use only - This product not cleared for use in the U.S.