




## Saliva•Strip HIV

- **Not available in the USA**


A rapid, oral-fluid test system for HIV antibodies. FOR PROFESSIONAL USE ONLY. Not available in the USA.

**Fast, Simple, Non-Invasive**

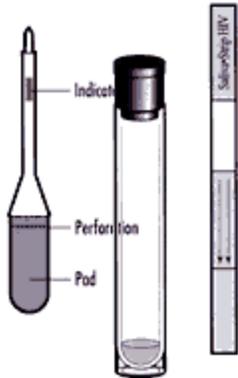

- No reagent preparation
- No timed steps
- No special training
- Built-in control
- Room temperature storage
- Results in minutes
- Sensitivity > 99.4%
- Specificity > 99.4%


### KIT COMPONENTS

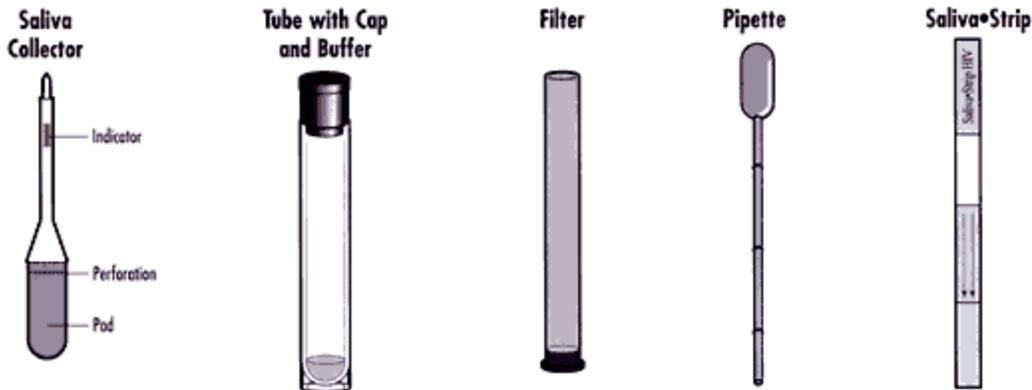

### General instructions for collection, processing, and testing of an oral fluid specimen with Saliva•Strip HIV

#### Collect The Specimen

1. Place collector under the subject's tongue. Leave in place until indicator turns blue.
2. Remove cap from tube

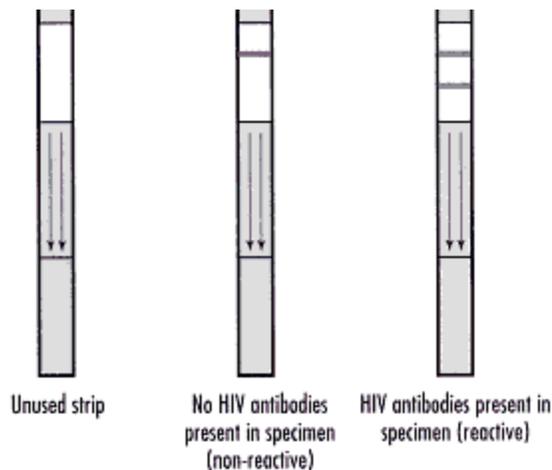

containing buffer. Insert collector pad-first into tube. Detach pad from stem of collector by vortexing. Insert and push filter down into tube.

### Test The Specimen

1. Transfer sample by pipette into test tube (not provided).
2. Drop a SalivaStrip, arrows pointing down, into the tube containing sample. Results available in 20 minutes, read within 2 hours.

### **Disclaimer**

© 1999 -2000 Saliva Diagnostic System