

# Uni-Gold HIV Rapid Test

For use with whole blood, serum, or plasma  
Store Kits: 2 - 30° C




- Check kit before use. Use only items that have not expired or been damaged.
- Bring kit and previously stored specimens to room temperature prior to use.
- Always use universal safety precautions when handling specimens. Keep work areas clean and organized.

This outline is not intended to replace the product insert or your standard operating procedure (SOP).



1. Collect test items and other necessary lab supplies.

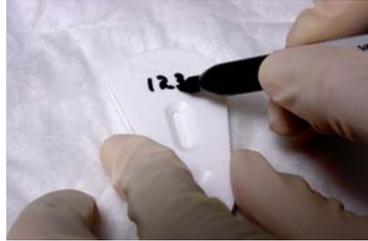

2. Remove device from package and label device with client identification number.



3. Collect specimen using the disposable pipette.



4. Add 2 drops (approx. 60µl) of specimen to the sample port in the device.



5. Add 2 drops (approx. 60µl) of the appropriate wash reagent to sample port.



6. Wait for 10 minutes (no longer than 20 min.) before reading the results.



7. Read and record the results and other pertinent info on the worksheet.

## Uni-Gold HIV Rapid Test Results

### Positive

**2 lines** of any intensity appear in **both** the **control** and **test** areas.



### Negative

**1 line** appears in the **control** area and no line in the test area.



### Invalid

**No line** appears in the **control** area. Do not report invalid results. Repeat test with a new test device even if a line appears in the test area.



