

Oraquick

This outline is not intended to replace the product insert or your standard operating procedure (SOP).

Procedure (Oral Fluid)

1. Check the expiration date. Do not use expired kits.
2. Set the reusable stand on a flat, level surface.
3. Tear open the foil pouch containing the test device and the developer vial. Remove the developer vial.
 - a. Label the device and vial with the appropriate patient/client identification.
 - b. Carefully uncapped the vial by gently rocking the cap back and forth.
 - c. Place the uncapped vial into the stand.
4. Instruct the patient/client to remove the test device from the foil pouch without touching the collection pad.
5. Check to see if the desiccant pack is present. If no desiccant is present, discard the unit.
6. Instruct the patient/client to swab completely around the outer gums with the test device, by gently wiping the porous flat pad completely across the upper and lower gums, one time around.
7. When the patient/client has finished swabbing the gums, insert the pad end of the test device all the way down into the vial. *Note: Be sure the result window faces forward so it can be read.*
8. Set the timer for 20 minutes. Do not exceed 60 minutes.
9. Read the results in 20 minutes.
10. Record results on the worksheet.