


Uni-Gold

This outline is not intended to replace the product insert or your standard operating procedure (SOP).


Procedure

1. Check the expiration date. Do not use expired kits.
2. Remove the test device from its protective wrapper.
3. Label with the appropriate patient/client identification.
4. Using one of the disposable pipettes supplied, fill with sample.
5. Holding the pipette over the sample port, carefully add two drops of sample (approx. 60µl).
6. Add 2 drops (approx. 60µl) of the appropriate wash reagent to sample port.
7. Wait a minimum of 10 minutes for reaction to occur and read the result. Results are stable for at least 20 minutes after addition of sample to the device.
8. Record results on the worksheet.


Interpretation of Test Results

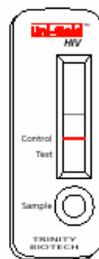

NEGATIVE
A line in the control region only indicates a negative test result.

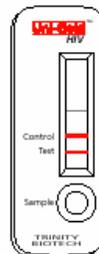

POSITIVE
A line in both the test and the control region indicates a positive result.

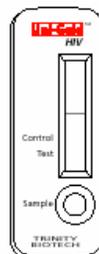

INCONCLUSIVE
No line appears in the control region. The test should be repeated with a fresh device. Irrespective of a line developing in the test region.